BAUERFEIND AG
 Triebeser Straße 16
 07937 Zeulenroda-Triebes
 GERMANY
 P +49 (0) 36628 66-20 00
 F +49 (0) 36628 66-29 99
 E info@bauerfeind.com
 bauerfeind.com

Rev. 0 – 2017-03_112760

VenoTrain® angioflow

Gebrauchsanweisung
 Instructions for use
 Kullanim Kilavuzu

MADE
 IN
 GERMANY

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en english

These instructions for use can also be downloaded from www.bauerfeind.com/downloads.

Dear patient,

Your physician has prescribed you the VenoTrain angioflow based on your diagnosis. The VenoTrain angioflow is the only compression stocking that can also be worn for incipient arterial occlusive disease. The inspection opening in the foot area allows your circulation to be checked. Should you own other compression stockings from the time before your latest diagnosis, these are no longer to be used in light of your current condition.

Thank you for placing your trust in Bauerfeind. By purchasing VenoTrain angioflow, you have selected a product that meets very high medical and quality standards.

The excellent workmanship, perfect fit, and use of materials that are particularly gentle on the skin give you the feeling of well-being that your legs need every day. For treatment to be successful, your product has to fit perfectly. This is why accurate measurements of your leg were taken at seven points and the product was custom made to fit these measurements. It therefore does not bear a CE marking (custom-made device in accordance with Article 3 No. 8 of the German Medical Devices Act).

We recommend wearing the VenoTrain compression stockings every day and consulting your physician on a regular basis. This will ensure successful treatment.

our instructions for use and care. They are based on years of experience and ensure that our medically effective products continue to work for a long time. After all, only aids that function perfectly will help you. Failure to follow the instructions can limit the warranty.

Disposal

The compression stockings can be disposed of in household waste.

For more details, please consult your physician.

Donning tips (Product with closed toe option)

- 1 Turn the stocking inside-out up to the heel (fold it over to the left). Pull the foot part over your toes and then over the upper part of your foot.
- 2 Fold the leg part back over and pull it over your heel at the same time. Make sure that the heel is in the correct position.
- 3 Now pull the stocking up gradually and distribute it uniformly until it is free of creases. Avoid pulling on the top band when donning the garment.

Using the VenoTrain glider/VenoTrain glider plus donning aid makes it easier to put on your VenoTrain compression stockings.

Version: 2017-03

1 Ratio between blood pressure in the ankle to blood pressure in the upper arm

2 Indicates the risk of personal injury (including health risks, accident risks and potentially also damage to property)

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Please read through all of these **instructions for use** carefully, because they contain important information about the use, indications, and contraindications, as well as tips for putting on and caring for your product. Please keep these instructions for use, so you can consult them later if any questions arise.

Intended use

- Long-term treatment of chronic venous insufficiency (CVI) in the case of initial PAOD, i.e. with an ankle-brachial index¹ of between 0.5 and 0.9 and an arterial pressure of 60 mmHg at the ankle.

Indications

- Chronic venous insufficiency according to the intended use in combination with incipient PAOD

Regular medical checkups of your condition and the fit of your stockings every 3 months will ensure successful treatment.

Where to apply the orthosis

Lower leg in the case of initial PAD.

Risks of using this product

⚠ Caution?

There are no known application risks if the product is used properly, provided your legs have been measured by a trained specialist and you have been taught the correct method for putting on the compression stockings.

Contraindications

- Advanced peripheral arterial occlusion (PAOD – circulatory disorders of the leg arteries [with an ankle-brachial index of under 0.5])
- Decompensated cardiac insufficiency (myocardial weakness)
- Septic phlebitis (bacterial inflammation of the vein)
- Phlegmasia coerulea dolens (acute and extreme venous occlusion with blue discoloration and heavy swelling of the leg)
- Material intolerances
- Paresthesia of the extremity (numbness and reduced pain perception with advanced peripheral neuropathy)
- Consult your physician in cases of weeping dermatoses (skin rashes)

The manufacturer will not accept liability – particularly product liability – in the event of failure to comply with the above contraindications.

Application instructions

- The initial fitting of the product must only be performed by medically trained specialists. The special notes on measuring according to the VenoTrain® angioflow measurement data sheet are to be observed with VenoTrain® angioflow
- Please note: Compression stockings are put on, not pulled on! Pulling on the stockings could overstretch them.

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- It is best to put on your stockings first thing in the morning, as soon as you get up.
- Wear rubber gloves (e.g. Bauerfeind gloves) to put on and take off your VenoTrain compression stockings as this makes it easier to distribute the material evenly over the leg.
- Your legs must be dry.
- Do not grasp the knitted fabric with sharp fingernails. Make sure that you take off all jewelry before starting.
- Protect your VenoTrain compression stockings against contact with oils, creams and ointments that are applied to legs for therapeutic or cosmetic purposes. We recommend that all creams, ointments, etc. be applied to the skin in the evening after you have taken off your compression stockings.
- Avoid damaging the knitted fabric (e.g. as a result of wearing unsuitable shoes) and do not cut off any loose threads or labels, as this could cause stitches made during production to unravel.
- If unexpected problems with the compression stockings do occur, please remove them as a precaution and consult a physician.
- Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (putting on and removal, care etc.).

Cleaning instructions

- Avoid dry cleaning and ironing.
- Prolong the life of your stockings with daily handwashing or machine washing on a delicate wash cycle (please note the care instructions on the label of your stocking).
- The product can be machine-dried provided a delicate cycle is chosen (refer to the information on the label). However, direct heat should be avoided (radiators, sunshine). When handwashing the stockings, we recommend drying them between two towels (do not wring).
- Only use a mild washing powder/liquid and avoid the use of washing additives like fabric softener, optical brighteners, or stain removers.

Please refer to the care instructions on the fabric label of your compression stocking.

Notes on reuse

VenoTrain compression stockings are intended for your treatment only. For this reason, do not pass your VenoTrain stockings on to other people.

Warranty

The legal provisions of the Federal Republic of Germany apply. Where relevant, the country-specific warranty regulations between the buyer and retailer apply. Please contact your medical retailer directly in the event of a potential claim under the warranty. Please do not make any alterations to the product yourself. Please follow

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