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en — english

Dear customer,
thank you for choosing a Bauerfeind product.
We would like to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ErgoPad ski & skate is a medical device for the feet. It is a supportive foot orthosis that corrects the shoe's internal volume, reduces lateral forces, and provides lateral support hindfoot during athletic activities. Suitable for: Downhill and cross-country skiing, ice hockey, inline skating.

Indications
The ErgoPad ski & skate can be used for prevention and concomitant treatment of sports-related overloading complaints:

- Non-specific overloading complaints involving the foot
- Achilles tendinitis
- Achilles tendonitis
- Knee complaints
- Knee complaints

It is also used for prevention and concomitant treatment of sports-related complaints involving the foot:

- Midfoot overloading (arch decreased / spaly foot or
- Mid pes cavus / arch high foot or

Risks of using this product
⚠**Caution:** Before using / applying a foot orthosis, you should consult a physician or specialist / on the risks involved.

The available internal volume of the shoe is not sufficient for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and/with shoes with inadequate lateral guidance in the case of open shoe types), problems relating to the foot orthosis, or if the foot orthoses are worn in shoes that do not meet the requirements for use.

Please remember that shoes and foot orthoses form a single functional unit.

- If the feet are positioned on the edges of the foot orthosis, blisters and calluses may form on the sides of the feet. The same symptoms may occur if excessive mechanical strain leads to holes or creases developing in the top cover and / or cushioning material, or causes them to break. Foot orthoses that are damaged in this way must no longer be used.

Contraindications

People with diabetes or rheumatoid foot deformities must seek medical advice before using this product to ensure optimal treatment.

Application instructions
Shoes suitable for wearing with foot orthoses must have sufficient internal volume for the foot and the selected foot orthosis (adequate shoe length / width and, if necessary, a removable, sufficiently thick cushioned sole, as well as lateral guidance for the foot orthoses in the case of open shoe types). An uncomfortable fit that restricts the foot must be avoided.

- It is recommended to use the shoe for the first time with the foot orthosis when the shoe is worn straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper increases the risk of pressure marks or blistering on the skin of the sides of the feet.

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by the treating physician.
The ErgoPad ski & skate is designed to be worn repeatedly by one user only.
Please regularly check your feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any questions, please contact a specialist or your physician.

Adjustment phase: As with many medical products, the ErgoPad ski & skate may feel somewhat strange at wear at first. We therefore recommend that you start by wearing the orthosis for around one hour a day and slowly increase the wearing period until wearing the orthosis feels comfortable. It should not take much longer than a week to adjust to wearing the foot orthosis. If you still find it uncomfortable to wear after this period, please consult a specialist.

Foot orthoses must be checked by a specialist after they have been worn for approximately half a year to ensure that they are still functioning properly.

Cleaning instructions
The ErgoPad ski & skate can be cleaned by hand using soap and lukewarm water. They should not be washed in a washing machine. Do not use steam cleaning. Use only a soft brush. Use warm water for cleaning. Do not use dry cleaning. The foot orthoses cannot be washed in a washing machine. Dry clean only. The product may be cleaned better by submitting it in case of warranty claims.

Part of the body this product is used for
The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty
The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the terms and conditions of the warranty. If you are dissatisfied with our handling, please contact our warranty department. Please refer to the instructions for use for more details.

- Warranty may be limited or excluded if the instructions on how to handle and care for the ErgoPad ski & skate have not been observed.
- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report
Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal
Once you have finished using the product, please dispose of it according to local specifications.

Modification instructions
Fitting: For an optimum fit, there should be a length allowance of approximately 1 cm between the foot and the foot orthosis and / or shoe, in a standing position. If the foot orthoses are not the right size for your shoes, you can cut them down or reshape them using appropriate tools. Please follow the instructions below.
Cutting:

- Take the inlays out of your downhill skiing / cross-country skiing / ice hockey boots or the inner soles where you wear your ErgoPad ski & skate.
- Align the heel sections so that they are positioned precisely on top of one another and draw around the inlay on your ErgoPad ski & skate. Make sure you do not cut the foot orthosis too small. You recommend cutting off less material than you do not cut the foot orthosis too small. It is possible that you will have to adjust it at a later date and try it on again after cutting.

4. Place the ErgoPad ski & skate in your shoes (foot orthoses for left / right feet are marked as such on the back).

Thermforming
In exceptional cases and having sought specialist advice, it may be necessary to use thermforming to adapt the foot orthosis to your foot:

- 1. After cutting down your ErgoPad ski & skate, preheat your oven to 90 °C / 200 °F.
- 2. Put the foot orthoses (with the top covers facing downward on a clean surface) in the oven for approximately 2 minutes.

Risk of burning! Wear protective gloves when taking the foot orthoses out of the oven.

- 3. As soon as you have taken the foot orthoses out of the oven, place them immediately in your shoes.

- 4. Put on your shoes straight away (make sure you are wearing socks or stockings).
- 5. Stand for 20 minutes with your feet shoulder-width apart and pointing forward. Your foot orthoses should now be adapted to the shape of your feet.

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