 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49(0) 36628 66-35 00
F +49(0) 36628 66-39 99
E info@bauerfeind.com
BAUERFEIND.COM

Rev.11 – 2021-02_103851

ViscoBalance®

Viskoelastische Einlagen
Viscoelastic heel cushions
Talonnettes visco-élastiques
Taloneras viscoelásticas

 **MADE**
 **IN**
 **GERMANY**

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoBalance is a medical device for the feet. It is a viscoelastic heel cushion to compensate for different leg lengths of 3, 5 and 10 mm. With the heel wedge, the product also offers relief of the Achilles tendon.

Indications

- Difference in leg lengths
- Achilodynia (tendinitis, bursitis subachillea)

Risks of using this product

Important Information

- **ViscoBalance must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.**
- ViscoBalance is a prescribed product that should be applied under a physician's guidance. ViscoBalance should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- The use of ViscoBalance during sports should be

discussed with the medical provider.

Contraindications

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed.

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.

Application instructions

Insert the ViscoBalance heel cushion in the shoe so that it lies flush with the heel section of the shoe.

Product care

We recommend washing ViscoBalance handwarm with a mild detergent.

Notes on reuse

ViscoBalance is intended for your treatment only and has been specifically selected according to your measurements. For this reason, do not pass the ViscoBalance on to other people.

Places of use

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over

many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.