AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

AchilloTrain is a medical product. It is a support for the treatment of Achilles tendon problems.

Important: The supplementary heel pad must be worn on the healthy foot to compensate leg length.

Indications

- Achillodynia (tendinosis, paratendinitis, bursitis subachillea)
- Haglund's deformity
- Chronic, post-traumatic or post-operative irritation (e.g. in the case of Achilles tendon ruptures)

Risks of using this product

- If you have been prescribed the AchilloTrain, please talk to your physician regarding the use and duration of use. They will also inform you whether the AchilloTrain can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the AchilloTrain as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The heel wedge may be removed from the product if necessary. To do so, please turn the support inside out and remove the heel wedge through the opening in the velour

pocket. If you remove the heel wedge, please make sure to also remove the compensating pad for the healthy foot.

- The effects of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the AchilloTrain – e.g. an increase in discomfort – stop using it and contact your physician without delay.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and circulatory disorders of the foot, e.g. Diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

A donning aid is included to allow for easier donning of the support. For donning, please follow the instructions provided in the quick reference guide.

- 1 Fold the upper part of the support outwards until the heel wedge can be gripped.
- 2 Pull the support over the foot until the heel is at the heel wedge.
- 3 Then pull up the folded-over part of the support taking care that the profiled insert is to the side of the Achilles tendon.
- 4 Adjust the fit of the support as necessary.

Do not forget: Insert the supplied compensation pad in the shoe of the healthy foot in such a way that it is flush with the heel area of the shoe.

Cleaning instructions

Note: never expose the support to direct heat, e.g. heater, sunlight, leaving it in your car, etc. This can impair the effectiveness of the support.

Please refer to the care instructions on the sewn-in label. This can be found on the upper edge of the support.

Please note the following:

1. Remove the heel wedge before washing. Wash the wedge by hand.
2. We recommend using a lingerie bag.
3. The support is not suitable for tumble drying.

Wash your support regularly to retain the compression of the knitted fabric.

To make a complaint, please contact the retailer that provided you with the product.

Note: only clean goods will be handled.

Notes on reuse

This product is intended for your treatment only. Do not pass the support on to other people.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the AchilloTrain have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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