



## Anziehhilfe für Bandagen – Hand und Fuß

### Sehr geehrte Kundin, sehr geehrter Kunde,



diese praktische Anziehhilfe spart Zeit und Kraft beim Anlegen Ihrer Bandage. Folgen Sie einfach den Abbildungen der Kurzanleitung. Lesen Sie bitte zusätzlich die Gebrauchsanleitung der Handgelenk-/Sprunggelenkbandage vollständig und sorgfältig durch.

### Anziehen/Anlegen



- 1 Legen Sie die Anziehhilfe mit der Spitze nach vorn vor sich hin. Dabei liegt die kürzere Seite der Anziehhilfe oben. Schlüpfen Sie mit der Hand/ dem Fuß zwischen den beiden Lagen der Anziehhilfe hinein. Bei der Hand bitte den Daumen außen lassen.

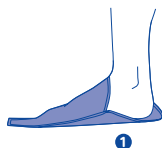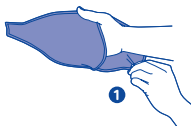

- 2 Schlagen Sie den oberen Rand der Bandage um und ziehen Sie diese über die Anziehhilfe bis die Bandage den optimalen Sitz erreicht hat. Schlagen Sie den oberen Rand zurück.

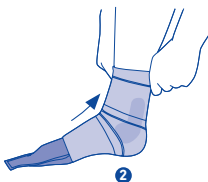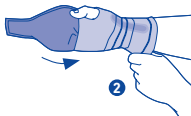

- 3 Ziehen Sie die Anziehhilfe nach vorn über die Finger/Zehen heraus. Falls nötig korrigieren Sie den Sitz der Bandage.

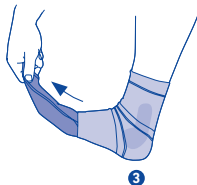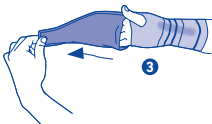

### Reinigung



### Entsorgung

Sie können das Produkt entsprechend den nationalen gesetzlichen Vorschriften entsorgen.

Stand der Information: 2017-01

# Donning aid for wrist and ankle



## Dear Customer,

Using this practical donning aid you can save time and effort when donning your wrist and ankle support. Please just follow the pictures of this short guide. In addition, please read through the instructions for use for the wrist and ankle support carefully and in full.

### Putting on/Donning



- 1 Position the donning aid with the tip pointing forward in front of you, so that the shorter side of the donning aid is on top.

Slide your hand/foot between the two layers of the donning aid. Concerning the hand: Leave your thumb outside.

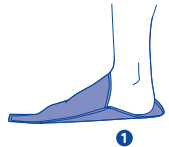
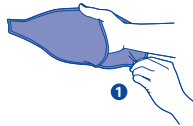


- 2 Turn over the upper edge of the support and pull it over the donning aid until the optimum fit is achieved.

Fold back the upper edge.

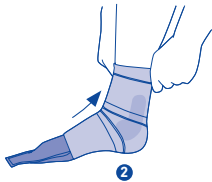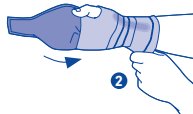

- 3 Remove the donning aid by pulling it out in a forward direction over the fingers/toes. If necessary, adjust the fit of the support.

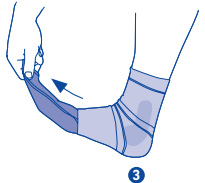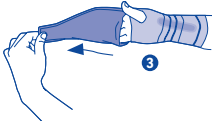

### Cleaning



### Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2017-01