**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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Polyamid (PA), Styren-Ethylen / Butylen-Styren-Blockcopolymer (TPS-SEBS), Elastodien (ED), Polyurethan (PUR), Elastan (EL), Viskose (CV), Polyester (PES), Polyoxymethylen (POM), Baumwolle (CO)

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

### Intended purpose

The GenuTrain S is a medical device. It is a support for the guidance and stabilization of the knee joint using lateral splints.

### Indications

- Joint instability (mild to moderate)
- Osteoarthritis of the knee
- Arthritis (e.g. rheumatoid arthritis)

### Risks of using this product

#### Caution\*

GenuTrain S must only be worn in accordance with the specifications contained in these instructions and used for the areas of application listed. To ensure the optimum fit of

the GenuTrain S, it is essential to check whether the product is adequately shaped. The GenuTrain S can be reshaped by a specialist<sup>1</sup> to suit the patient's anatomy if necessary.

Use in combination with other products, e.g. as part of compression treatment (involving wearing compression stockings), must be discussed beforehand with your physician.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.

No product liability is accepted in the event of improper use.

Do not allow the product to come into contact with greasy or acidic agents, ointments and lotions.

Do not make any changes to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Do not put the GenuTrain S on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the GenuTrain S and, if necessary, have the size of your GenuTrain S checked.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

### Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness and excessive heat build-up
- Varicosis with chronic venous insufficiency (also known as varicose veins)
- Impaired sensation in the leg due to neurological abnormalities
- Impaired blood flow in the leg (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support

### Application instructions

#### Putting on the GenuTrain S

#### A – Support, B – Velcro straps, C – Guide channels with joint splints

- 1 First loosen the four ends of the Velcro straps of the GenuTrain S without pulling them out of the slits. 2 Grip the tightening straps at the sides with your fingers and pull the support over the affected knee until the kneecap is enclosed by the massage ring (pad), without any pressure being exerted on it. The Velcro ends of the straps should now be on the outside of the leg. 3 By pulling the straps you can stabilize the knee comfortably. 4 Fasten all four of the strap ends.

#### Taking off the GenuTrain S

- 5 Undo the straps. Grip the support at the level of the calf. Pull the support off in a downward direction.

## Cleaning instructions

**Note:** Never subject the GenuTrain S to direct heat (e.g. heaters, sunlight, in your car, etc.!) This could damage the material, which can impair the effectiveness of the GenuTrain S.

The sewn-in label of the GenuTrain S, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the upper edge of the support.

Remove the Velcro straps. Fasten the ends back onto the straps to avoid any damage. Pull the joint splints upward and out of the guide channels. The joint splints can be wiped off with a damp cloth. The white hinge projection cover which encases the hinges of the joint splints helps to protect the knitted fabric and should not be removed. Wash the support separately in a lingerie bag with a mild laundry detergent at 30 °C (no woolite). Do not run the spin cycle on your washing machine at speeds higher than 500 rpm. Wash the GenuTrain S regularly so that the knit fabric retains its compressive properties.

Once the support has dried, guide the joint splints back into the guide channels from above, in line with the markings. The joint thus point toward the hollow of the knee. Fasten the straps onto the eyelets of the joint splints.

If you have any complaints, please contact your medical retailer only.

Please note that only clean goods can be dealt with.

### Notes on reuse

This product is intended to be fitted to one patient only.

### Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain S have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

### Material content

Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Elastodiene (ED), Polyurethane (PUR), Spandex (EL), Viskose (CV), Polyester (PES), Polyoxymethylene (POM), Cotton (CO)

## Specialists

### General notes

- Check the optimal and individual fit of the GenuTrain S after the first fitting. Make corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect the medical and therapeutic effectiveness of the product, thereby voiding any product liability.
- The GenuTrain S is intended for the treatment of one patient only.

### Shaping the joint splints:

- 1 Remove the Velcro straps.
- 2 Take the joint splints out of the guide channels on the support and remove the hinge projection cover. Check the fit of the joint splints on the patient. Pay attention here to any particular anatomical conditions (e.g. fibular knobs).
- 3 If necessary, adjust the joint splints with a hot-air gun at 120–140 °C.
- 4 Wear protective gloves while you are doing this.
- 5 Attach the hinge projection cover. Then place the support on its side so that the guide channel is directly in front of you. The arrow marking on the joint should be pointing upward, while the curve of the joint splints points toward the hollow of the knee.
- 6 Insert the joint splints into the guide channel through the large upper slits at the back.
- 7 Pull the guide channels outward and upward for easier positioning at the tightening straps.

### Fitting the straps:

- 1 Sort the straps according to length and width. Arrangement: wide straps for the thigh area, narrow ones for the lower leg, longer straps for the back of the support, shorter ones for the front (side with Bauerfeind B). Place the support in front of you with the front facing downward.
- 2 The straps are always attached at the upward-facing eyelets, starting from the inside of the support (take right / left version into account). Push the two longer straps with the straight ends and Velcro fastening upward through the eyelets. Fasten the ends together just after the eyelets.
- 3 Then guide the rounded strap ends with Velcro fastening upward through the eyelet on the opposite side. Fasten the strap ends just after the eyelet.
- 4 Turn the support over. Now follow the same procedure with the shorter straps.

<sup>1</sup> A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports and orthoses which are relevant to you.

\* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

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