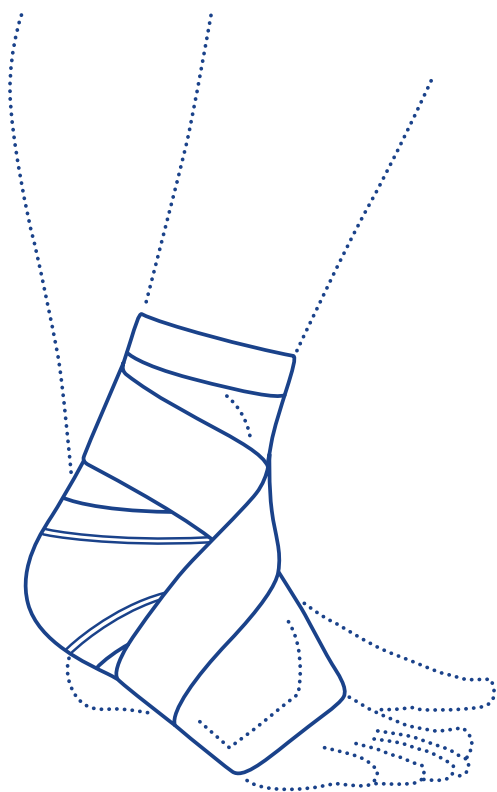
MalleoTrain® S

SPRUNGGELENKBANDAGE

ANKLE SUPPORT

CHEVILLÈRE

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Dear customer,
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MalleoTrain S is a medical product. It is a support for stabilization of the ankle joint thanks to a functional strap system.

Indications

- Ligament insufficiency
- Supination prevention
- Slight sprains

Risks of using this product

- The benefits of this product are felt particularly during physical activity.
- Please remove the support during longer periods of rest.
 - Once you have been prescribed the MalleoTrain S, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
 - Avoid contact with ointments, lotions, or substances containing grease or acids.
 - Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
 - If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
2. Impaired blood flow in the leg / foot (e.g. with diabetes mellitus)
3. Impaired lymphatic drainage and soft tissue swelling of unknown origin.

Application instructions

Putting on the MalleoTrain S

- Fold the top part of the support outward until you can reach the heel. Pull the support over the foot until it reaches the heel.
- Then unfold the turned-over part of the support and pull it up until the heel section is positioned correctly.
- The start of the strap is located on the inside of the foot.  When winding the strap, make sure that it sits without folds. To ensure the optimum fit of the strap, your lower leg must be perpendicular (at a right angle) to the foot during all the following steps.

- Guide the strap under the sole of your foot toward the outside of the foot.  By pulling the strap, you can generate the stabilization that's comfortable for you.
- Place the strap diagonally across the instep and back around the ankle mortise  and then guide the strap diagonally forward again to the inside of the foot  
- Pull the strap under the sole of your foot toward the outside of the foot and over the instep  and fasten the end of the strap to the section of the strap underneath it. 
- Loosen the strap and attach the end of the strap onto another part of the strap. Take hold of the upper edge of the support and turn it down.

Taking off the MalleoTrain S

- Loosen the strap and attach the end of the strap onto another part of the strap. Take hold of the upper edge of the support and turn it down.

Cleaning instructions

Close all the Velcro fastenings. Please use mild detergent and a lingerie bag. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoTrain S have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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Bauerfeind USA, Inc.
BAUERFEIND AG