Liebe Patientin, lieber Patient,

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Dear Patient,

thank you for placing your trust in Bauerfeind. By purchasing the VenoTrain glider donning aid, you have selected a product that meets very high medical and quality standards. The excellent workmanship provides you with the convenience you require to put on your compression stockings quickly and accurately, day after day.

Please read the instructions for use carefully. Please keep the instructions for use safe. Should you have any questions, please consult the physician providing your treatment, your medical retailer, or contact us directly.

Intended purpose

The VenoTrain glider is an accessory to a medical device. It makes it easier to put on medical compression stockings for the lower and upper extremities.

Indications

- Restricted mobility

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

The VenoTrain glider donning aid must only be used in accordance with the instructions in this user guide and the intended use given.

Please do not put strain on the arm or leg until you have fully removed the VenoTrain glider and properly put on the compression stocking. There are no known risks if the product is used properly.

Contraindications

are not known

Use on the leg

Putting on the product

- Put the foam surface – point forward – on the floor in front of you.
- Place the VenoTrain glider on top, with the white side facing upward. Make sure that the blue loop protrudes slightly beyond the tip of the foam surface.
- Place your toes in the marked semicircle.
- then push your foot gently forward with slight pressure. Your foot is now in the donning aid.
- Pull on your compression stocking as usual until you reach your calf. Make sure that the blue tape on the back of the lower leg sticks out from the stocking.
- a) For compression stockings with closed toe: Grip the blue tape and pull the VenoTrain glider slowly and steadily out of the compression stocking while gently moving your toes. b) For compression stockings with open toe: Pull the VenoTrain glider out of the stocking with the loop at the front.
- Using the Bauerfeind rubber gloves, bring your compression stocking into its final position on the respective stocking style.

Important: After use, turn the VenoTrain glider to its correct side (with the loop pointing outward).

Taking off the product

- Put the pull tape in the VenoTrain glider and slide it in your foot. The blue side of the donning aid is up.
- Fold up your compression stocking to the VenoTrain glider and place the edge or stay-up tape over it.
- Pull the compression stocking over the heel and instep towards the tip of your foot. When you remove the stocking, your VenoTrain glider is inside it.

Use on the arm

- Put the pull tape in the VenoTrain glider and slide your arm in it. The blue side of the donning aid is up.
- Now pull the arm sleeve over the hand up to the upper arm and push it into the right position (seam in the crook of the elbow).
- Pull out the VenoTrain glider by the tab, check the position of the stocking and adjust this again if necessary. You can also hook the tab of the VenoTrain glider onto something secure. Then you can pull out the VenoTrain glider.

Cleaning Instructions

The VenoTrain glider is hard-wearing and absolutely easy to clean. Simply wipe it with a moist cloth, allow to air dry at room temperature, and you're done. Folded up and tucked away in the storage bag, the very light-weight VenoTrain glider fits any carrying bag. The way how to use the VenoTrain glider is illustrated on the following pages.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disclaimer

For use of the VenoTrain glider, the following applies: Follow the advice of the medical specialist as well as all instructions in this brochure or its online version, including any excerpts (as well as texts, images, graphics etc.). The VenoTrain glider is exclusively intended for putting on and taking off the medical product. Never wear the medical product together with the VenoTrain glider.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Polyurethane (PUR), Polyester (PES)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

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