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## dear customer,

## Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

### Intended purpose

GloboTec Junior is a medical device for the feet. It is an orthopedic shell foot orthosis suitable for treating foot deformities in children and adolescents until their body growth is completed.

### Indications

GloboTec Junior foot orthoses effectively support the treatment of foot deformities.

The indications addressed include:

- High-arched foot (pes cavus), also with flattening
- Skew foot (pes valgus)
- Pediatric skewed flat foot (pes planovalgus), which cannot be compensated for
- Congenital flat foot / arch-decreased foot (pes planus)
- Pathological leg axis development in children with foot deformity

### Risks of using this product

Before using / applying a foot orthosis, you should consult a physician or specialist\* on the risks mentioned above.

If there is not enough space inside the shoe for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and width (for shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing).

If the feet are positioned on the edges of the foot orthoses, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excess mechanical pressure leads to holes or creases developing in the top covers, wearing material or orthotic cores, or to the breakage of these. You must **stop** wearing foot orthoses if they are damaged in this way! Creases in stockings or pantyhose must be pulled flat before the shoes are put on, as creases of this kind can cause blisters and pressure points.

### Contraindications

Weak pediatric skewed flat foot (pes planovalgus), compensated for when standing on toes

### Application instructions for the wearer of the foot orthoses

The foot orthosis is exclusively intended for use by growing children following clinical assessment by a specialist.

Combining the aid with other medical products (e.g. compression stockings, supports) may only be considered after having sought specialist advice. Shoes suitable for wearing with foot orthoses must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length / width, removable, sufficiently thick cushioned sole, adequate lateral guidance in the foot orthoses in the case of open shoe types). An uncomfortable fit that constricts the foot must always be avoided.
**ⓘ**

When selecting the correct size of foot orthoses, the packaging information serves only as a guide. Please check the fit of the foot orthosis. If you are unsure which size to choose, please contact your specialist.

The foot orthosis should always lie flat at the midfoot (b1) and hindfoot (b2) – without tilting.
**ⓘ**

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your physician.

The foot orthosis should be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet.
**ⓘ**

If you are unsure about anything during the fitting or regarding the fit, please contact your local specialist.

Please regularly check your child's feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact a specialist or your physician.

### Adjustment phase

As with many medical products, foot orthoses may feel somewhat strange to wear at first. We therefore recommend that your child starts by wearing the foot orthosis for 1–2 hours at a time and slowly increases the wearing period until it feels comfortable. Normally, it should not take longer than a week to adjust to wearing the foot orthosis. If your child still finds it uncomfortable to wear after this period, please contact a specialist. The aid supports your child while they are growing and therefore requires particular attention. Please ask your specialist about appropriate check-up intervals.
**ⓘ**

### Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to avoid moisture to evaporate. The foot orthoses are not machine-washable. All surface materials facing the foot can be wiped if required, using a mild cleanser and a sponge or cloth. After cleaning, any residual soap must be thoroughly rinsed away with water. The orthoses must be left to air dry and no external heat source should be used (heating, hair-dryer, direct sunlight). Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of top cover materials.

### Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected foot orthosis.

### Notes on reuse

The product is designed to provide care for an individual patient / individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper use of the product.

### Warranty

Foot orthoses are designed for daily use and as such they are subject to natural wear when used on a regular basis.

Slight discoloration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, such as moisture levels and sock / stocking materials. This is not covered by warranty. The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

You can dispose of the product in accordance with national legal requirements.

### Material content:

Thermoplastic elastomer (TPE), Polypropylene (PP), Polyester (PES)
Polyurethane (PUR), Polyethylene (PE), Ethylene vinyl acetate (EVA)

 – Medical Device  – DataMatrix Unique Device Identifier UID

Version: 2020-04

<sup>\*</sup>A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic orthoses.

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