AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Dear Customer,

These assembly instructions contain important information on the adjustment options of the product.

Please also read and observe the instructions for use carefully. If you have any questions, please contact your physician or our technical service department.

⚠ The SofTec Coxa must be fitted to the patient individually and in a qualified manner by a trained specialist². This is the only way the full effectiveness and ideal wearing comfort of the orthosis¹ are guaranteed. This product is intended to be fitted to one patient only.

Application instructions

Application and removal of the SofTec Coxa should be practiced with the patient.

Application and removal of the SofTec Coxa:

- Place SofTec Coxa around your pelvis. Position the orthosis snugly on your iliac crests. Take care that the side bar runs laterally, centered on the thigh (Fig.1).
- For basic positioning, insert your fingers into the finger pockets on both parts of the closure belt and secure by evenly pulling. The right half of the closure belt is placed on top of the left half of the closure belt and adhered with Velcro (Fig.2). First remove fingers from the left and then from the right finger pocket.
- Tension straps ensure the medical-therapeutic effect and the correct positioning of the SofTec Coxa.
- To fix the SofTec Coxa, take the tension straps, which are passed around the back, in both hands and pull them evenly towards the front (Fig.3). Take care here that the straps run along the lower part of the pelvic shells and attach both ends of the straps with Velcro tape to the lower part of the front fastener (Fig.4). The ends of the straps can also be attached to one another with Velcro tape, as required.
- For easier handling of the SofTec Coxa, always refasten the opened straps loosely right away with the Velcro tape. This should be practiced with the patient.
- There must be basic tension of the straps on the pelvic shells so that the SofTec Coxa cannot slip. Straps that are too loose lead to a loss of medical efficacy.

Leg harness:

- Position the outside leg harness centered on the thigh and place the leg support around the thigh from the back. Insert your fingers into the finger pockets on both halves of the fastener provided for this purpose (Fig.5).
- Secure the closure belt by snugly attaching the Velcro tape from the side of the closure belt coming from the inside with the fastener coming from the outside (Fig.6).
- Now fasten the two front straps on the leg harness (Figs. 7 and 8).
- The rear supporting straps do not need to be opened when attaching and removing the SofTec Coxa. The rear straps have already been adapted to your thigh circumference by the orthotist. If you have to adjust their position, take care that the leg harness runs laterally to the thigh. To correctly readjust the rear straps, loosen the rear and front straps at the same time. Position the leg harness centered on the thigh and pull evenly on the lower front and lower back straps. Repeat this procedure for the upper straps.
- If using the orthosis for both hips, attach the second leg harness to the first one.

Removing the SofTec Coxa:

- Open the front straps of the leg support first and attach these loosely to the straps with the Velcro tape and then open the closure belt on the leg support. Then open the straps on the pelvic harness and attach them loosely to the straps with Velcro tape. Now the closure belt on the pelvic harness can be opened. Keep the SofTec Coxa tight on the side on which the joint / leg shell is located.
- For easier handling of the SofTec Coxa, always refasten the opened straps loosely with the Velcro tape.

Procedure for the initial fitting of SofTec Coxa (only by a trained specialist):

1. Assembling the pelvic harness

- The SofTec Coxa consists of two plastic pelvic harnesses on which prefabricated joint supports are mounted for the right or left side of the pelvis, as required. A right and a left joint support are included with the product. Depending on whether the right or left hip requires care, one or the other of the two joint supports will be set aside. Should both hips require care, then use both joint supports.
- Insert the joint support (Figs. 9 and 10) for the required side from the inside behind the pelvic harness (Figs. 9 and 10).
- Secure this joint support with 4 plastic clips (Fig. 11).
- The rivet connector fixes securely with an audible clicking sound.

Mounting the hinge on the pelvic harness

- To adjust pelvic width, the supplied flat connecting tab, with the 2 Velcro strips facing inwards (1 Velcro strip faces outwards), is latched into the head element of the pelvic harness (Fig. 12). This connection tab allows five-step adjustment for pelvic width.
- This connection tab is positioned in the correct fixed setting if the pelvic harness is positioned on the iliac crests and the side bar runs centered along the longitudinal axis of the thigh.
- Now screw on the hinge with the two M5x10 screws (Figs. 13 and 14).
- Only tighten the hinge so that the side bar can be pushed into the support for its entire height.

Positioning the pelvic harness on the iliac crest

- To correctly adjust the hinge rotation point to the anatomical rotation of the hip joint, slide the side bar for its entire height. The left and right pelvic harnesses must be positioned symmetrically on the body (Fig. 15). Loosen the "click connector" on the connection tab for this purpose. Join the pelvic harness with the connection tab at the final position of adjustment.
- Fix the setting using the plastic rivet to both pelvic harnesses (Fig. 16).

2. Combining the fixed elements with the knitted fabric

- The pelvic frame with the joint attachment is first inserted into the pocket provided on the knitted fabric portion. Then position the opposite side analogously (Fig. 17).
- For the 1st fitting it is advisable to fix the plastic support using the inside Velcro tape.

3. Adapting the knitted fabric material

- Position the pelvic harness snugly and prefix the fastener (Fig. 18).
- To individually adapt the knitted fabric support material open the circumference widening panel on the back. Adapt this to the individual circumference size by cutting it to shape (Fig. 19).
- Take care that the knitted fabric support material is applied tightly on the complete pelvic harness in the case of a pre-fixed fastener.
- Remove the orthosis from the patient and shorten the inside padding centered on the connection tab (Fig. 20).

4. Assembling the thigh shell with the leg support

- Before you fix the leg shell to the lower side bar, slide the joint protector onto the lower side bar with the open side facing the front.
- Open the zipper on the shell case.
- Mount the leg shell incl. shell case on the lower end of the side bar using two M5x6 screws so that the length of the bar can be adjusted.
- Apply the pre-assembled orthosis to the patient as described above. Ensure the correct pelvic positioning and the correct orientation of the side bar in relation to the axis of the hip joint. Now adjust the outside leg shell to its correct height. The lower end of the leg shell is located on the outside (laterally) at around the height of the supracondylar support (Fig. 21).

5. Adapting the leg support

- To adapt the leg support you have to measure the circumference at the middle of the thigh.
- For pre-adaptation, shorten the velour according to the individual circumference. When closing the fastener, take care that the adjusting strip is not creased (Fig. 22).

Secure the leg shell incl. shell case with the leg support

- Apply the Velcro strip on the shell case onto the outside of the Velcro loop strip on the leg support (Fig. 23).
- The position of the shell case depends on the side under treatment.
- Remove the leg support closure belt from the velour and adapt the knitted fabric support material to the individual circumference by cutting it to shape. Reapply the leg support closure belt.
- Pass the straps through the plastic rings and loosely attach them to the straps with the Velcro tape (Fig. 24).
- Further adaptation of the leg support on the patient is described in Section 7 "Adapting the orthosis".

6. Adjusting the hinge

- Before adjusting the hinge, please open the leg shell.
- The hinge can be used on the right and the left. There is a scale on the side bar above the symbols R and L to set the flexion angle. The 70°-90° limit setting for the flexion range is shown left and right of the center axis (blue line) for the side under treatment. 60° flexion is on the center axis.
- On the hinge disc under the cap there is a scale to set the extension angle (Fig. 25).
- Remove the cap, limiting pins and extension scale (Fig. 26).
- Each time the hinge disc is rotated by 90°, the abduction / adduction angle shifts by 6° from the zero position. Set the abduction / adduction angle as required for the patient. Fix this setting with a limiting pin on the center line (60° flexion).
- For final limitation of movement, place the extension scale back on the hinge disc. Fix the movement limitation using the 2 limitation pins for flexion (side bar scale) and extension (hinge disc scale) according to the angle required and the set abduction and adduction. Place the limitation pin to set the extension angle and then position the limiting pin to set the flexion angle by displacing the limiting pin from the center line (60° flexion), as required.
- Then secure the hinge with the cap. Fix the joint protector on the leg shell with the adhesion point on the hinge. Now pull the joint protector over the hinge (Fig. 27).

7. Adapting the orthosis

- Place the orthosis around the body so that the point of rotation of the hinge coincides with the greater trochanter.
- Place the pelvic harness on the iliac crest (Fig. 28).
- Ensure correct positioning when securing the orthosis.
- Slide the fingers into the finger pockets on both halves of the closure belt provided for this purpose and pull evenly to the front.
- Press the left half of the closure belt onto the abdomen and the right half as far over it so it is possible to attach the Velcro tape (Fig. 29).
- When attaching the Velcro tape, first remove the fingers from the left and then from the right finger pocket. Take the two straps in both hands and pull evenly to the front until the required force is applied.
- Mark this length on the strap.
- Cut the two straps to this length (Fig. 30), and attach to the new end with the Velcro strip. Now attach the two straps to the large fastener with Velcro tape (Fig. 31).
- Correct the height of the leg shell by sliding the side bar into the right position. Now tighten the screws (Fig. 32). The lower end of the leg shell is located at around the height of the supracondylar support on the lateral side.
- Place the leg support around the thigh.
- Slide the fingers into the finger pockets provided for this purpose and pull evenly to the front.
- Overlap the halves of the closure belt until they can be attached with the Velcro tape (Fig. 33).
- Pull the straps to the back and front until the required force is applied.
- Mark this length on the straps.
- Cut the straps to this length (Fig. 34), remove the ends of the Velcro tape from the cut-off strap and attach to the new end.
- Now attach the straps to the large closure belt with Velcro tape (Fig. 35).

If treatment is needed on both sides, proceed according to points 1 and 4 to 7 in the chapter Procedure for the initial fitting of SofTec Coxa.

When applying the orthosis for the first time, the optimal and individual fit must be checked and corrected as necessary by trained specialist personnel. The correct application should be practiced with the patient.

SofTec Coxa Assembly and returns information

⚠ Important Note: Final fixation of the pelvic shell is not reversible. Therefore, a return / refund cannot be provided.

- Place connecting plate for appropriate size adjustment at the two halves of the pelvic shell.
- At a suitable size secure the connecting tabs of the two pelvic shells with the plastic rivets.

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Information status: 2023-08

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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